

UriCap Female: A Non-Suction based Solution for Women with Urinary Incontinence.

Covered by Medicare Part B and Many Insurance Plans

UriCap is soft and flexible providing comfort and convenience for women with urinary incontinence without using suction. By forming a natural seal, it offers a low-risk solution for nighttime or continuous use in bed. UriCap is especially beneficial for wheelchair users.

Ideal Candidates:

Women who:

- Seek a comfortable nighttime solution for urinary incontinence.
- Require continuous use while in a wheelchair or bed.

Application Requirements:

- Self-application users should be able to independently reach the area while lying in bed, spreading their legs and labia wide enough with one hand and applying the device with the other.
- or
- Have a caregiver available to place or assist with placement if needed.

Key Considerations:

- Using UriCap is all about learning to position the device to create a seal. After application, users should remain lying in bed for 1 hour to allow the seal to strengthen. Applying the device before bedtime is recommended.
- Gradually increase wear time for women with heightened sensitivity in the area.
- Always stabilize the device with a leg strap or a catheter securement device.
- Treat any irritation or active infections (e.g., UTIs, yeast infections) before using UriCap.
- UriCap is not suitable for women with protruding prolapses of stage 2 or higher.

